# EXTRACTOR OF IMPLANTS AND SCREWS

## SPECIFIC USES OF THE EXTRACTORS

### EXTRACTOR TYPE 1 (SIS-9140)

Designed to extract screws with crowned or deformed connection.

Universal for hexagonals of 1.20, 1.25, 1.27, 1.30, unigrip, Torx, etc.

Suitable for screws, pillars of healing, etc.

### EXTRACTOR TYPE 2 (SIS-9140)

Designed to extract screws with the connection completely impaired.

Used when the extractor type 1 does not hold or drag the affected area.

### EXTRACTOR TYPE 3 (SIS-9141A)

Designed to remove implants threaded M1.6.

Universal for most of implants from the market.

It should not exceed more than 100 N cm torque.

For higher torques use extractor 4.

### EXTRACTOR TYPE 4 (SIS-9141B)


Designed to remove implants threaded M1.8 y M2.

Universal for most of implants from the market.


It is advisable to not exceed more than 200 N cm torque.

Can also be used for implants M1.6 thread if previously the extractor type 3 has been used.


## INSTRUCTIONS FOR USE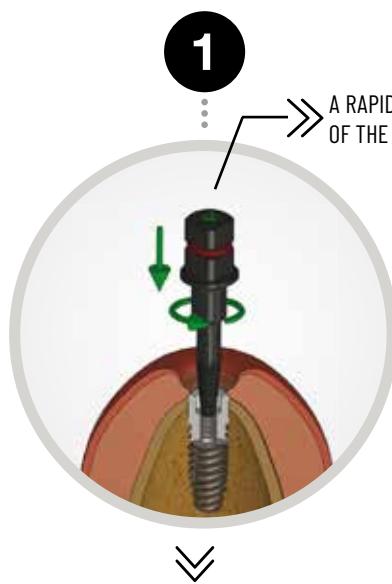

Insert the extractor manually to straight position and aligned with the implant and threading.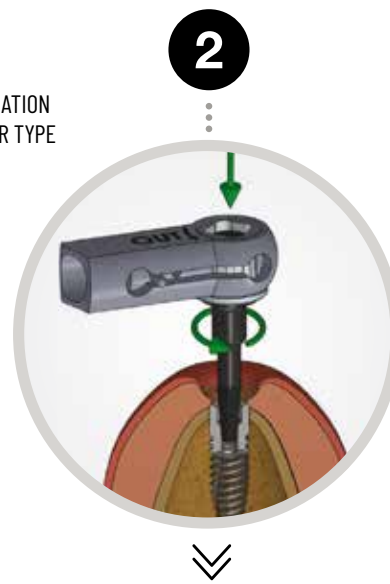

Attach the ratchet through the juicer and screw left clockwise, putting pressure on the ratchet.

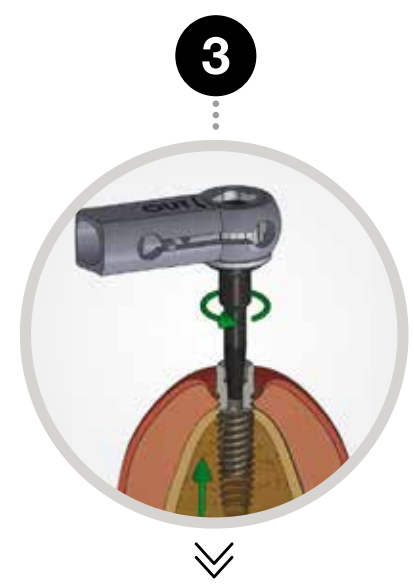When the extractor has anchored in the thread of the implant, it is anger turning progressively.

SIS-9139

» Implant and screw extractors can also be used with the SIS-9139 manual ratchet wrench.

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